# CAS 1 / CS 1 - Montagehinweis

# CAS 1 / CS 1 - Installation Instructions

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
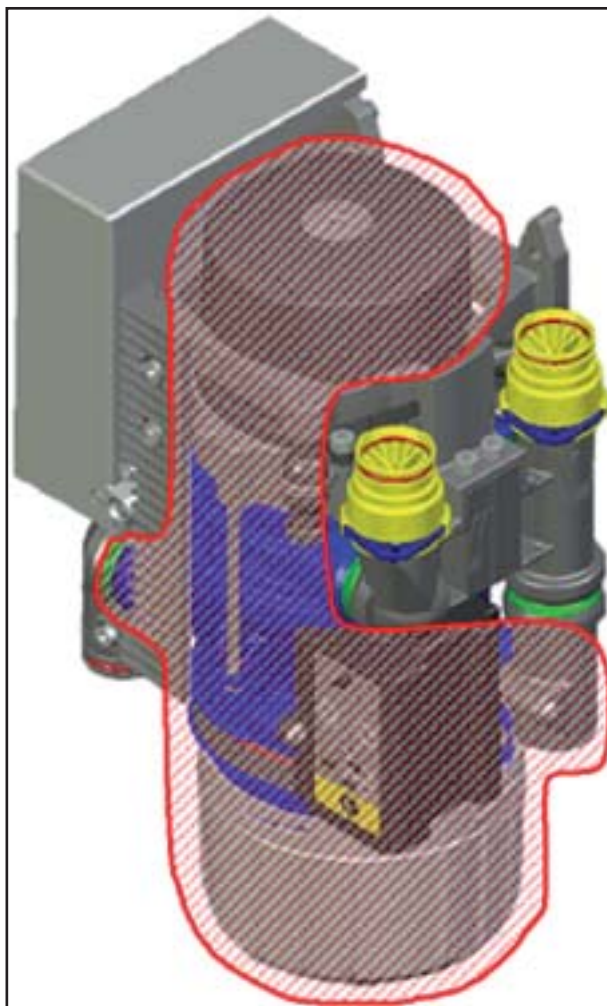
During operation the elastic suspensions absorb the vibration.

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In order for the dampers to absorb the vibrations properly a gap of about 3 - 5mm around the CAS 1 / CS 1 is necessary.

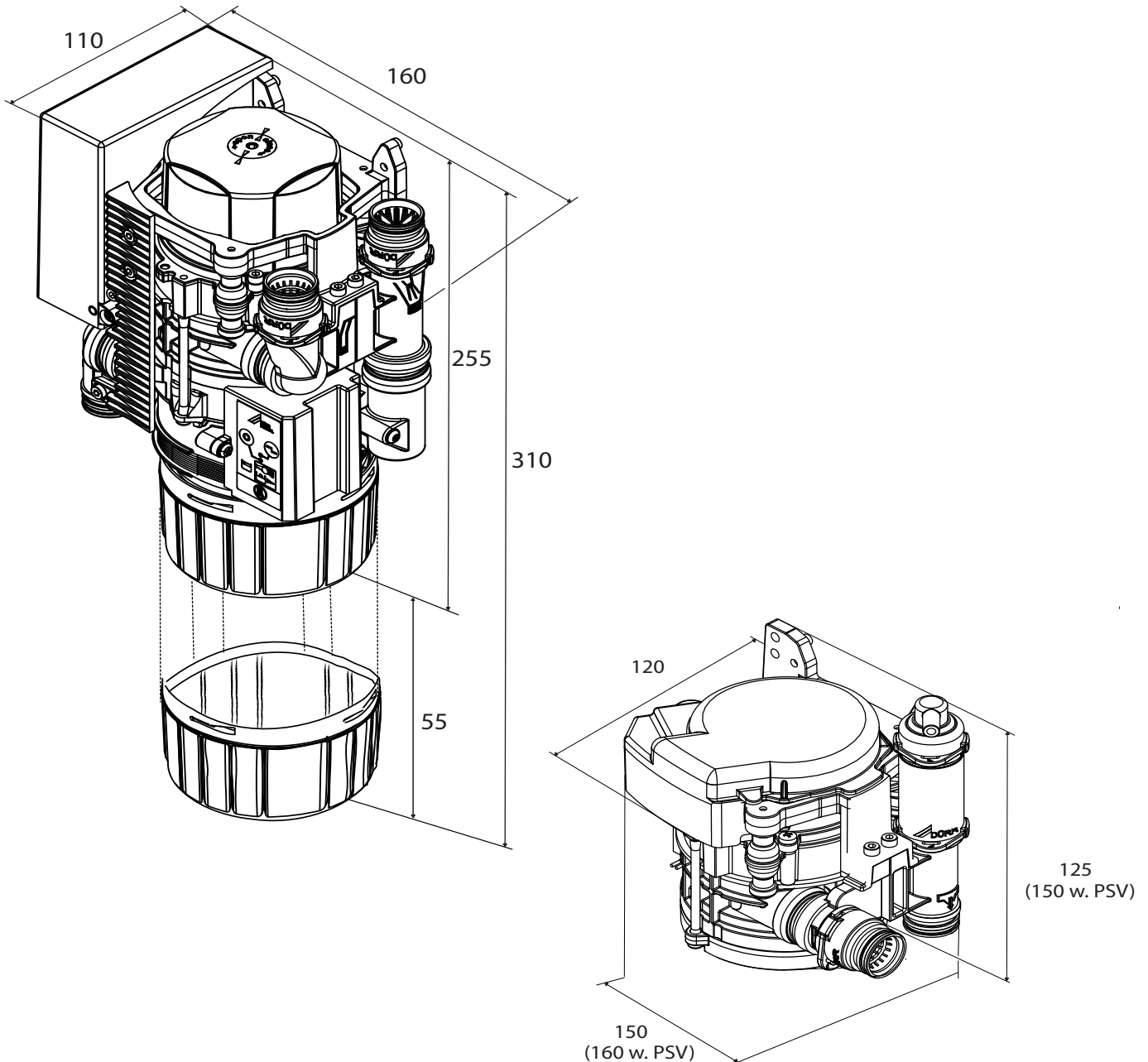**Abmessungen CAS 1 / CS 1 Combi Separator (Maße in mm)**  
**Dimensions CAS 1 / CS 1 Combi Separator (Measurements in mm)**

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When assembling the CAS 1 Combi Separator / CA 1 with GF 1 always leave room (55mm) below the CAS 1 for changing the amalgam container.

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
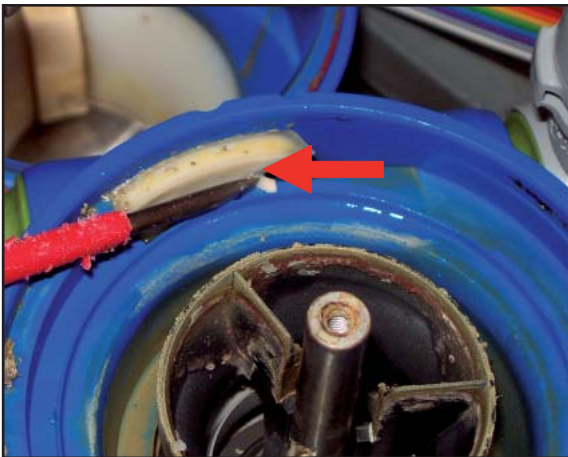
Foreign particles (tooth fragments) in the separation or the centrifuge drum can cause noise.

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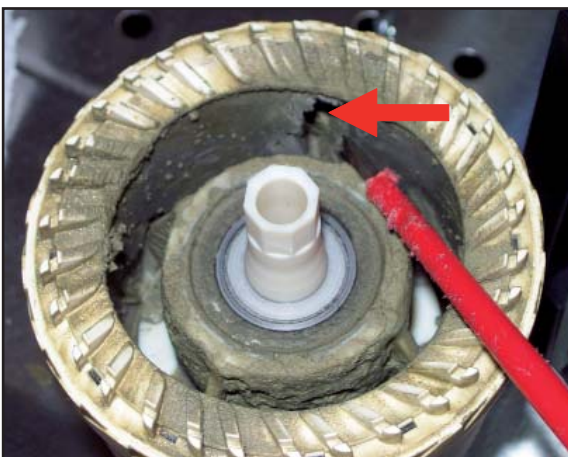We suggest using a Dürr Connect fine filter (0725-040-00) in the suction part and a coarse filter (0700-702-52) in the spittoon valve drain.

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Sediment in the amalgam centrifuge can cause vibrations.

Sufficient rinsing with water or through a rinsing unit can avoid this.

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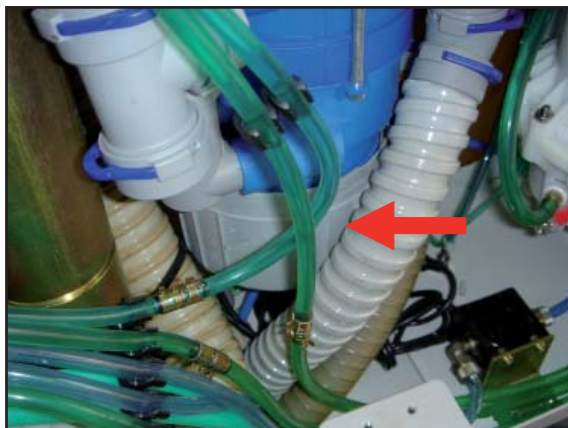
Do not fix cables or hoses between the vibration damper and centrifuge housing!

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Do not fix cables or hoses to the vibration damper!

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
Ensure that no cables or hoses touch or press against the centrifuge housing!

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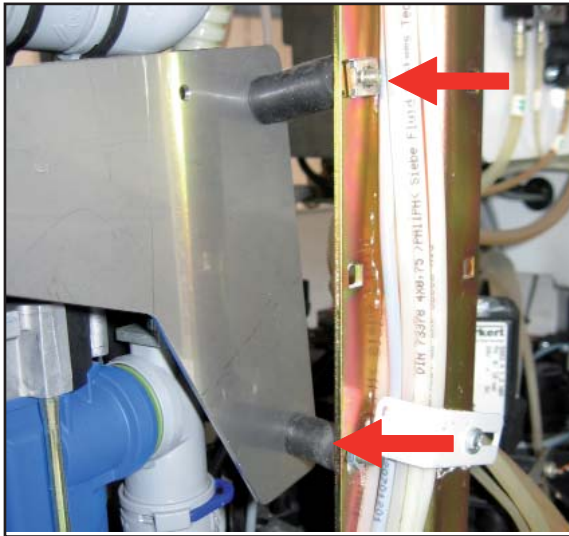Allow enough distance between the cables and the motor (vibration area) and to the mounting plate.

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Do not use any additional vibration dampers  
between CAS 1 / CS 1 mounting plate and  
dental unit.

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The CAS 1 / CS 1 should not have any contact  
with the mounting plate or other fixed parts in-  
side the dental unit.

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If there is limited room the black safety rings  
(0700-700-09) can avoid the CAS 1 and the  
DürrConnect parts touching.

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## Correct installation:

- CAS 1 / CS 1 should be fixed into the dental unit without additional vibration damper. If it necessary use additional mounting plate
- No contact or pressure with hoses, cables, motor, centrifuge housing or the amalgam container
- Spittoon valve drain with an integrated siphon and a float valve for ventilation (with extra flat securing ring)
- Fine and coarse filters available
- Free suspension
- Ventilation via float valve
- Ensure enough rinsing water is used!  
(we recommend using rinsing unit Article-Nº: 7100-250-50 or 7100-260-50)

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The Rinsing Unit Vario can be mounted onto the frame of the CS 1.

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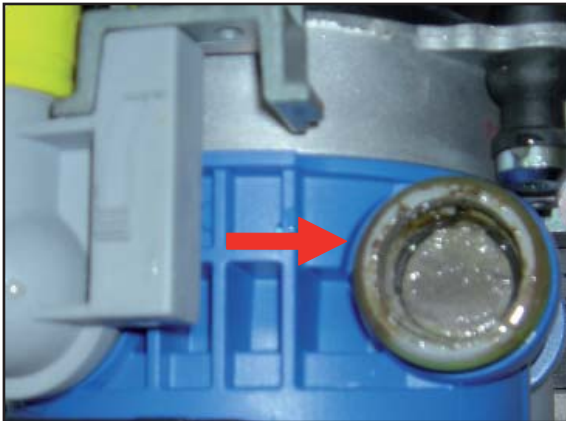
The Rinsing Unit must be mounted vertically and at least the same height or higher than the CS 1, CAS 1, Spittoon Bowl Valve or CA 1.

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
Defective or dirty non-return valve at the outlet of the separation unit. No rinsing water was used! (see chapter 3.2)

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Abrasion of the seal between the centrifuge drum and separation stage! (see chapter 3.3)

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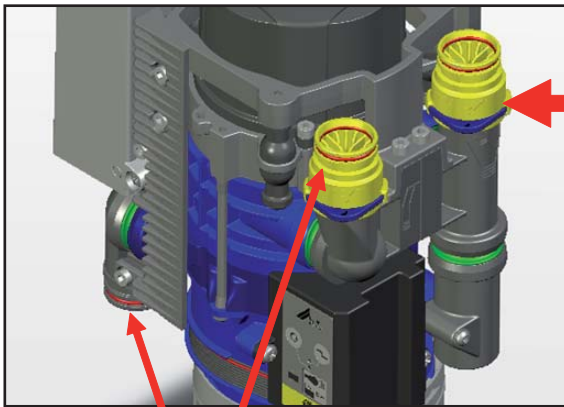Remove DürrConnect part 7117-420-26. Cover up the drain and suction hose connection with a blind cap.

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Check the sealing and valve membrane of the vibration damper for contamination and clean if necessary.

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
In general please use the new Dürr Connect vibration damper with non return valve:  
Art.-Nº: 7117-420-43E

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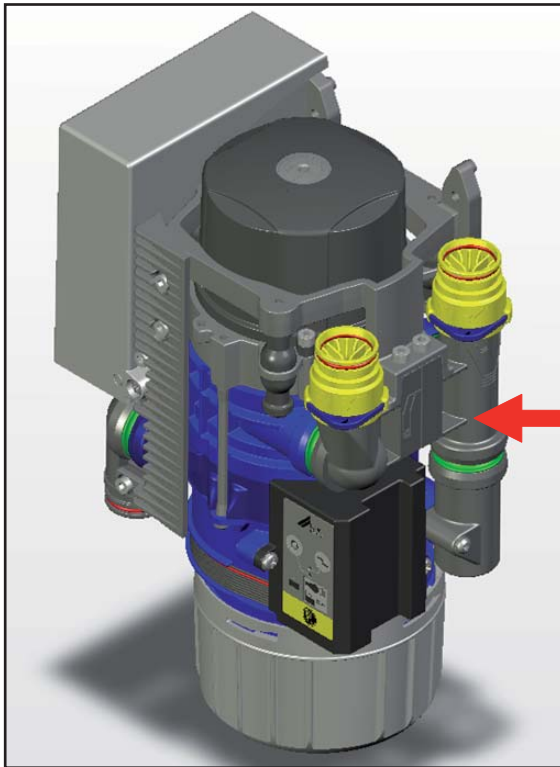### 3.2 Negative pressure measurement at the separator drain outlet

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The negative pressure measurement at the outlet of the separation stage can be used to test the non return valve. If the negative pressure is above 5 mbar (suction machine in use) then the non-return valve needs to be exchanged. In some cases also the housing seal.


Connect the negative pressure measuring instrument to the drain outlet of the separation stage

Negative pressure > 5mbar → exchange the DürrConnect vibration damper with non-return valve Art.-Nº: 7117-420-42E

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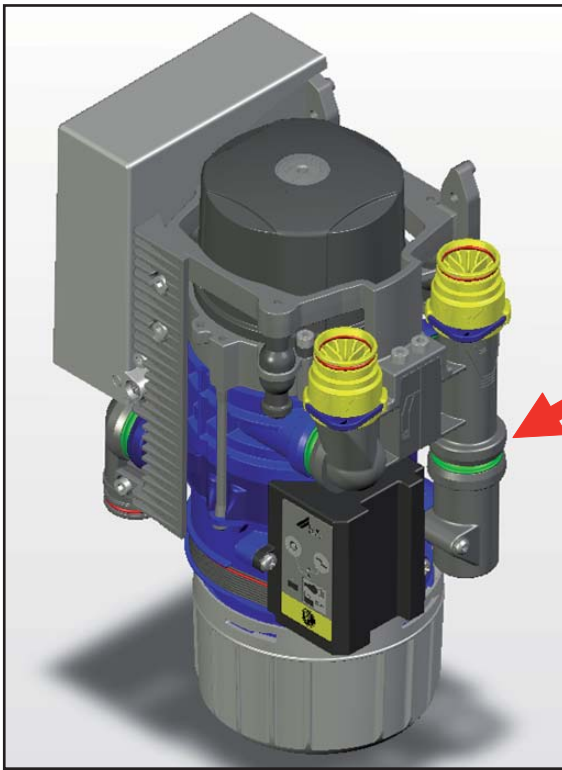### 3.3 Negative pressure measurement at step-seal

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Service-Kit CAS 1 Art.-Nr.: 7117-980-30

The negative pressure measurement at the entrance to the amalgam separation stage is used to check the seal between the separation and amalgam stage. If the negative pressure is above 5 mbar (suction machine in use) then the seal needs to be exchanged.

Connect the negative pressure measuring instrument at the inlet of the amalgam centrifuge  
 Negative pressure > 5mbar → exchange the vibration damper with non-return valve

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No or not enough ventilation at the outlet of the spittoon bowl.

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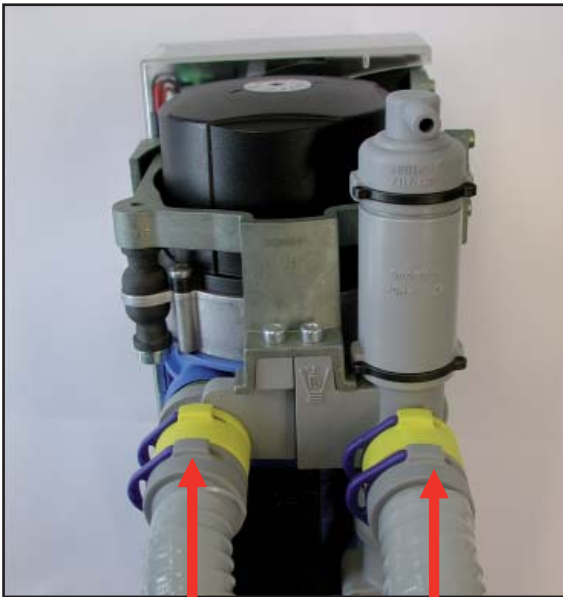
The yellow DürrConnect filter is polluted or blocked.

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The drain is blocked by waste, hose siphon is too large.

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Use the flat, black securing rings (Article-Nº: 0700-700-09E) for the float valve. Install the new yellow filter, no double hose syphon!

Install the fine filter (Article-Nº: 0700-700-28E) at the suction inlet and the coarse filter (Article-Nº: 0700-700-18E) at the spittoon inlet.

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