

Product information

Disposable filter for suction systems



0725-041-00 Disposable filter (12x)
0725-041-60 Disposable filter (36x)

Product description

The disposable filter from DÜRR DENTAL is designed to hold back impurities and particles (e.g. tooth fragments).

This protects the downstream separation and suction systems, such as the CAS 1 and the suction unit.

To ensure that full suction performance is delivered and the required hygiene standards are met, the disposable filter must be replaced once a week with a genuine DÜRR DENTAL replacement part.

The disposable filter is classed as a one-way filter and must not be washed out and reused repeatedly, as this could damage the filter fabric and cause the downstream components to no longer be protected.

The grip with integrated filter bar makes filter changes very straightforward.

Used disposable filters must be collected and disposed of by a licensed disposal company.

Installation location

- Hose manifold
- Pre-filter in the suction line (in the treatment unit)

Materials


- Filter: HDPE
- Fabric: PA
- Filter bar: PP

Mesh size

0.35 x 0.75 mm

Replacement interval

Weekly

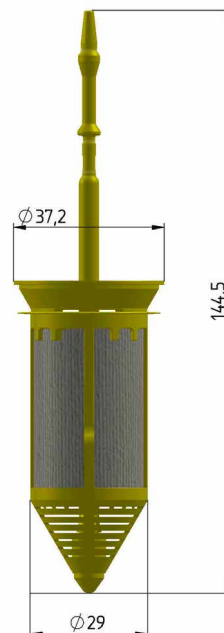

DÜRR DENTAL SE
Höpfigheimer Str. 17
74321 Bietigheim-Bissingen
Germany
www.duerrdental.com
info@duerrdental.com

Date: 2018-03

**DÜRR
DENTAL**
THE BEST, BY DESIGN