

EN

Installation Place Selection Valve 7560-500-71E

Use place selection valve (1) (without solenoid valve) only in combination with a pneumatic 3/2 multi-way-valve (2) in suction systems.

A 3/2 multi-way-valve (2) is par example in Dürr manifold Comfort inside.

Alternatively install a 3/2 multi-way-valve additional in the unit.



Take careful note of the fitting position!

A - from hose manifold

B - to suction unit

See direction on place selection valve!



Use protective gloves if you are installing the place selection valve as a modification to a treatment unit that is already in operation.



The membrane is vacuumregulated, also use never compressed air!

